Instructions for use Intensiv MarginFinisher

P235/04/2009

DE

GB

FR

IT

ES

Intensiv MarginFinisher

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- The tip of the instrument is perfectly aligned with the handle's main axis so as to avoid strenuous effort of the hand during application.

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Practical tips for usage

- Intensiv MarginFinisher is applied after the tooth cavity is prepared with rotating diamond instruments.
- Temporary red surface of the working ends, which is resistant to abrasion during the first applications, marks the side without diamond-coating.
- Proceed with light movements avoiding excessive pressure.

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Recommendations

- Disinfect the diamond instruments separately from non-stainless steel instruments, such as polishers and abrasives.
- Use only cleaning/disinfection solutions that provide corrosion protection, and strictly observe the concentrations and reaction times recommended by the manufacturer.
- Sterilization must be performed with validated procedures. If possible, use a single-pulsed or fractionated vacuum autoclave and subvacuum drying.
- Chemical sterilizers may also be used. Hot air sterilizers are not suitable for diamond instruments.
- Avoid temperatures above 180°C which may affect the durability of the instruments.

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