Instructions for use Intensiv PerioDiaCurette

P236/04/2009

DE

Intensiv PerioDiaCurette

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GB

Intensiv PerioDiaCurette

Diamond currettes for the delicate and final root surface planing and molar and premolar furcations.

Indications

- Delicate removal of root surface concretions.
- Delicate root surface planing.

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Practical tips for use

- Intensiv PerioDiaCurette is used after the root surface has been treated with a traditional curette.
- Proceed with light movements, without excessive pressure.

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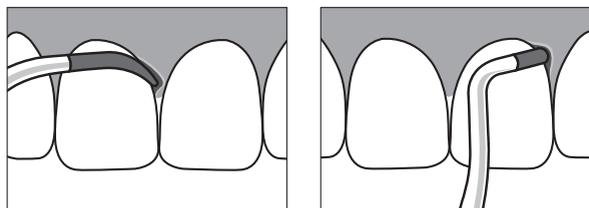
Recommendations

- Disinfect the diamond instruments separately from non-stainless steel instruments, such as polishers and abrasives.
- Use only cleaning/disinfection solutions that provide corrosion protection, and strictly observe the concentrations and reaction times recommended by the manufacturer.
- Sterilization must be performed with validated procedures. If possible, use a single-pulsed or fractionated vacuum autoclave and subvacuum drying.
- Chemiclave sterilizers may also be used. Hot air sterilizers are not suitable for diamond instruments.
- Avoid temperatures above 180° C which may affect the durability of the instruments.

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