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Intensiv Diamant-Rotierende Instrumente

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Instructions for use

Intensiv Diamond Rotating Instruments

(Trade name: Intensiv DiaRotary, Intensiv ZirconCut)

Product description

- Diamond abrasive rotating instruments for dentistry.
- Stainless steel with diamond coated abrasive working part.
- Diamond grit-sizes: 8µm (orange), 15µm (yellow), 25µm (white), 40µm (red), 50µm (gold), 80µm (blue), 106-106µm (silver), 106-125µm (green), 125-150µm (black).
- Shapes: ball, inverted cone, pear, cylinder, flame, cone, torpedo, football, wheel, lenticular, special shapes.
- Available in FG (Friction Grip), FGL (Friction Grip Long), FGM (Friction Grip Mini) and RA (Right Angle) according to ISO 1797.
- To be used with contra-angle handpieces which are medical devices suitable for the intended purpose and which have a standardised interface for FG or RA rotating instruments.
- Reprocessable
- Packed with single or multiple instruments (quantity indicated on packaging)

Intended use

Abrasion of natural (enamel, dentin and root cement) and artificial (e.g. composite, ceramic, cement) dental material for reducing, contouring, finishing and/or polishing in dentistry.

Application details

- Preparation and finishing for conservative and prosthetic dentistry.
- Separation of crowns and bridges made of zirconia ceramic (with Intensiv ZirconCut) or other dental materials.

Indications

For conservative treatment of tooth defects caused by caries disease and/or other defects (for example destroyed restoration) or prosthetic tooth reconstruction following trauma or other diseases (for example following endo treatment).

Contra-indications

The instrument is not indicated for application on any other substance (for example bone) other than dental material.

Patient target group

All kind of patients needing restoration of dentition from a dental medical professional.

Intended users

Clinical application by dental medical professionals.

Clinical benefits

Contribution to treatment of tooth defects caused by caries, trauma or other diseases for improving teeth function/occlusion.

Performance characteristics

Abrasion of natural and artificial dental material, feasible through the variety of shapes, dimensions, surface coating and diamond grain sizes, for the preparation of cavities and abutments, the modelling and finishing of restorations, and the separation of crowns/bridges.

Specifications for use

- Lock the instrument by inserting it as deeply as possible into the chuck of the FG (Friction Grip) contra-angle handpiece or until safety click is noticed with the RA (Right Angle) contra-angle handpiece. Check that the FG or RA contra-angle handpiece firmly holds the instrument.
- Only use FG or RA contra-angle handpieces which are maintained according to their specifications, and which are fully conform to their defined performance.
- Place the instrument close to the surface to be treated and set the contra-angle handpiece at the required working speed. Then start applying the instrument on the surface to be abraded.
- Activate water spray during the entire instrument application (minimum 50 mL/min) for cooling and instruments cleaning.
- Provide additional water spray cooling for Intensiv ZirconCut and instruments exceeding working part diameter ISO 020 and working part length of 10 mm, and diamond grain-size smaller or equal to 50 microns.
- Carefully modulate application load and rotating speed depending on the instrument used (dimension, shape and diamond grain-size), on the dental material to be abraded and to the type of treatment performed.
- It is recommended to use the entire length of the working part to allow for lower contact pressure and a uniform usage of the instrument.
- Apply intermittently, to allow for a more thorough rinsing of the instruments and removal of residues.
- Once the preparation has been completed, remove the instrument from the treated area and stop immediately the rotating movement.
- In case the instrument is no longer usable, dispose of it with care in dedicated puncture resistant containers, to prevent any contamination.

Precautions

- Ensure that the instrument is completely locked in the chuck of the FG (Friction Grip) contra-angle handpiece or RA (Right Angle) contra-angle handpiece.
- Ensure that there is consistent water flow, regulate in case of water shortage.
- For safety purpose perform treatments under rubber dam, and wear gloves and safety glasses.
- It is recommended, for better view, to use magnifying glasses
- Select the appropriate diamond grit-size and shape depending on the clinical application.
- Carefully ensure that contact load and speed are modulated during treatment according to type of instrument used and type of treatment performed, to avoid overheating, damage or breakage of the instrument, and consequently to avoid damage to the pulp and to the treated surface.
- Avoid using the instruments for drilling, as cooling water spray would not be able to reach the enclosed portion of the instrument, leading to overheating and instrument coating damage.

Reprocessing and maintenance

- Instruments are packed non-sterile. They must be, cleaned, disinfected and sterilized prior to first use on the patient and cleaned, disinfected, and sterilized after each use.
- Full cleaning, disinfection, sterilization, checking and maintenance instructions can be requested at info@intensiv.ch or visualized through the QR-code at the bottom of the page.
- Decontaminate the instruments immediately after use by immersion in a disinfectant solution.
- Cleaning, disinfection and sterilization must be carried out according to a validated procedure.
- Clean the instruments and remove possible debris after each use (with soft brush or ultrasonic bath), to maintain their abrasive properties.
- Check for possible damage; dispose of oxidized, eccentric or deformed instruments, and instruments where the diamond-coated working part is totally or partially worn-out.
- Sterilize the instruments using steam sterilizers. Hot air sterilizers are not suitable for diamond coated instruments.

Warnings

- Damaging of the diamond coating may occur if abundant water spray is not applied during usage
- Breakage of the instrument may occur if torque, jamming or levering actions are applied during usage.
- Overheating, damage or breakage of the instrument, and possible consequent damage to the pulp and to the treated surface may occur if contact load and speed are not carefully modulated during treatment, according to type of instrument used and to type of treatment performed.
- Possible damage to the pulp and to the treated surface caused by overheating may occur if the diamond-coated area of the instrument is fully or partially worn-out. Periodically check that the diamond coating is uniform, intact, and sufficiently present on the instrument.
- Overtreatment to the tooth may occur when using instruments for longer than requested by the therapy, specifically with coarse diamond grit size.
- Detachment of the instrument may occur if the instrument is not inserted as deeply as possible into the chuck of the FG (Friction Grip) contra-angle handpiece or until safety click is noticed with the RA (Right Angle) contra-angle handpiece.
- Loss of traceability of the instruments occurs if their packaging, with indication of reference and lot number, is not kept until all instruments of the packaging are used up.

Additional information

In the unlikely case of a serious incident caused by an instrument please contact the manufacturer Intensiv SA in Switzerland. Any serious incident that has occurred in relation to a device should be reported to Intensiv SA and to the competent authority of the state where the user and/or patient is established.

Liability

Intensiv SA declines any liability for any use of the instrument other than that specified in the instructions for use and in case of negligence by the user. The user is responsible for checking the instrument before use and for checking that the foreseen clinical application is in line with the instrument intended use.

Symbols:

Spray-cooling

Package unit

Intensiv SA
Via al Molino 107
6926 Montagnola
Switzerland

Tel. +41 91 986 50 50
info@intensiv.ch
www.intensiv.ch

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Instrumentos rotatorios de diamante Intensiv

(Trade name: Intensiv DiaRotary, Intensiv ZirconCut)

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